

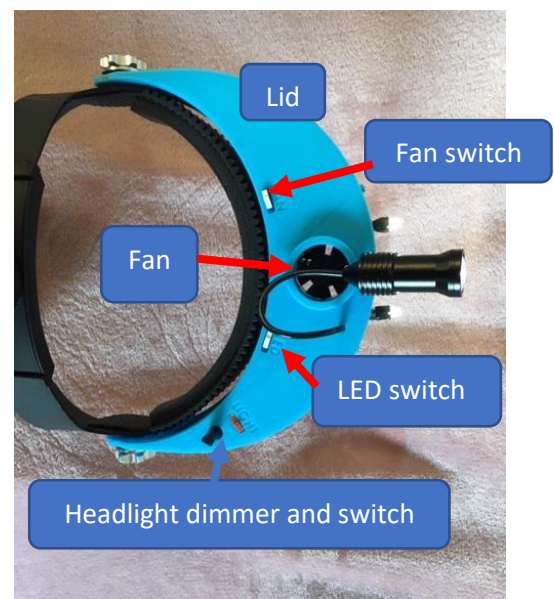
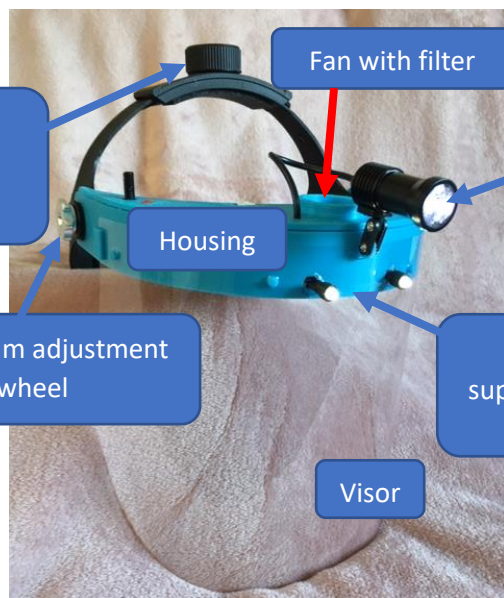



OrangeDot Face Shield Version 2A Quick Start Guide 27.08.2020

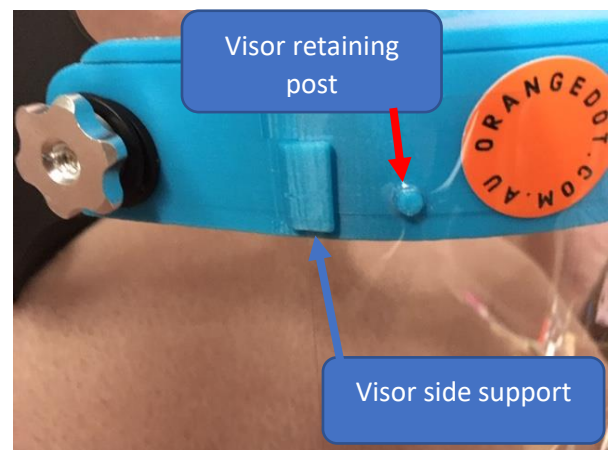

1. Please unpack the device carefully and ensure that the yellow battery isolation tabs are removed after opening the lid. The lid can be opened by lifting the crescent shaped corners of the lid from the housing.
2. Please carefully attach the transparent visor to the housing retaining posts, ensuring that the LEDs are threaded through their holes first. The edge of the visor should be tucked into the side visor support.

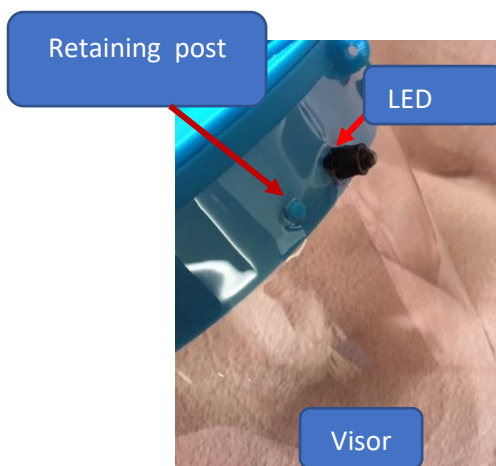

3. Adjust the vertical and horizontal headband wheels to ensure that the shield is held firmly and comfortably on your head.
4. You may raise the housing and visor by loosening the aluminium adjustment wheel.
5. Turn on the fan and LED switches. Use the dimmer switch for the headlight

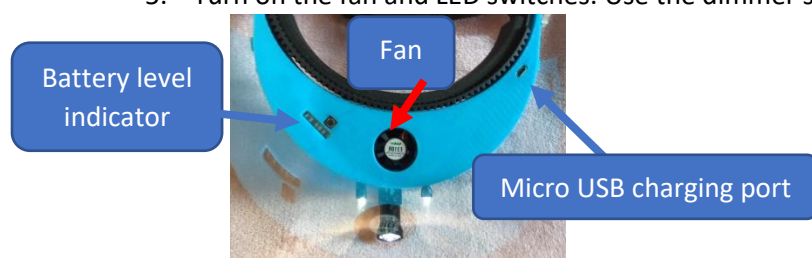

Please do **not** rest the face shield on the visor- it may fall and the visor will develop crease lines

For further information, please refer to the Operations Manual that came with your face shield. Manuals, Frequently Asked Questions and Instruction Videos are available at www.orangedot.com.au.