



## OrangeDot Respirator Quick Start Guide 18.12.2020

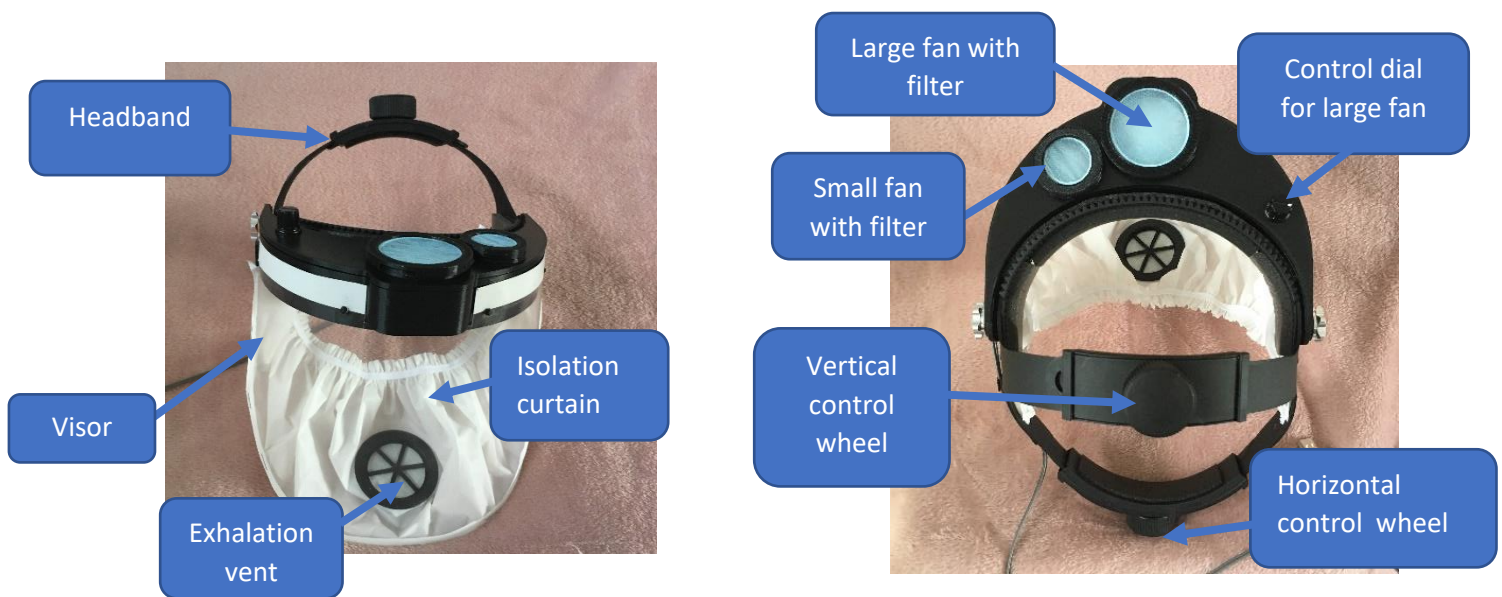

- Check that the free ends of the isolation curtain are attached to the Velcro type retaining strips on the inside of the headband. This will help to maintain a seal around the face.
- Attach the USB power cord to the supplied power bank. The small fan will start immediately, and the green LED status light will power on.
- Turn the control dial on the lid to power up the large fan. The blue LED status light will power on and get brighter as the power to this fan is increased.
- Gently insert your head inside the face shield chamber.
- Adjust the vertical and horizontal headband black control wheels, so that the shield fits comfortably on your head.
- Pull the plastic sheet over the front half of your face and tug the elastic band backwards, so that it forms a seal around your face. Remember that this seal will not be airtight.
- The Aluminium wheel on the side of the faced shield is non-functional in this version.

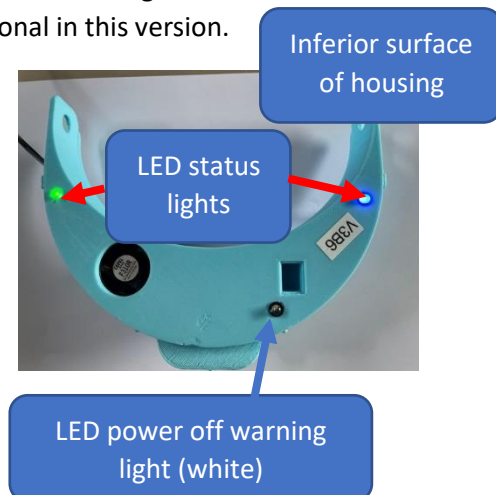

- Cutting the power supply to the shield or the energy depletion of the battery will result in a warning flash of the white LED light and an alarm noise. Please remove the device and check your connection or replace the battery.
- Any questions? Please read the comprehensive operations manual and frequently asked questions supplied with your device or contact us: [admin@orangedot.com.au](mailto:admin@orangedot.com.au).